KULZER VIVIDA

WHITENING

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EN Instruction for Use

Cosmetic Product - for patients under dental supervision and for dental professionals

Intended purpose

Kulzer Vivida Pre-Whitening Toothpaste

The Pre-Whitening Toothpaste is used to remove superficial discolouration and soft plaque. It can be applied to the teeth with a toothbrush to clean them optimally, increase the pH value in the mouth and prepare them for the tooth whitening process.

Kulzer Vivida Home Whitening Syringe

The 6% HPS gel is used as a hydrogen peroxide-containing bleaching agent in Kulzer Vivida Home Whitening Syringe. It is provided encapsulated as HPS Hydrogen Peroxide Superior in order to obtain a sufficiently stable product. The HPS Hydrogen Peroxide Superior dissolves quickly in oxygen radicals and water. During the bleaching process, the resulting free radicals attack the double carbon compounds in organic molecules, which in most cases are the cause of tooth discolouration.

Kulzer Vivida Tooth Conditioner

The Tooth Conditioner is a water-based gel with a mint flavour and contains hydroxyapatite and potassium nitrate. Kulzer Vivida Tooth Conditioner reduces sensitivity and optimizes the whitening result of the teeth by remineralising hydroxyapatite.

Indications

For correcting and whitening tooth colour.

Contraindications

The use is contraindicated for:

- pregnant and breastfeeding women
- children under the age of 18
- known hypersensitivity or allergies to mint flavour or any of the other ingredients
- those currently treated for a serious illness

Potential side effects

Hypersensitivity to the product or its ingredients cannot be ruled out in individual cases.

Contents

Kulzer Vivida Pre-Whitening Toothpaste

Package sizes/delivery forms

- Tube 50 g

Composition:

- Aqua, sorbitol, hydrated silica, glycerin, erythritol, lithium magnesium sodium silicate, sodium hexametaphosphate, glycolipids, cellulose gum, xanthan gum, phenoxyethanol, sodium saccharin, wintergreen extract, mint/eucalyptus flavour, CI 19140, CI 42090

Kulzer Vivida Home Whitening Syringe

Package sizes/delivery forms:

- Syringe 3 ml

Active Ingredient:

- 6% Hydrogen Peroxide

Composition:

PEG200, water, hydrogen peroxide, poloxamer, T-butyl alcohol, etidronic acid, mint flavouring, potassium hydroxide, EDTA, sodium stannate

WARNINGS

- Contains hydrogen peroxide.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- Keep out of reach of children.
- Avoid exposure of known sensitive teeth.
- Do not eat, drink or smoke while wearing the tray.
- Harmful when swallowed.

Kulzer Vivida Tooth Conditioner

Package sizes/delivery forms:

- Tube 50 g

Active Ingredients:

- 2% Hydroxyapatite
- 0.1% Potassium Nitrate

Composition:

- Water, hydrogenated starch hydrolysate (sorbitol), glycerin, erythritol, lithium magnesium sodium silicate, hydroxyapatite, potassium chloride, potassium nitrate, phenoxyethanol, xanthan gum, tetrasodium pyrophosphate, tartaric acid, mint flavour, sodium saccharin.

Colour

Kulzer Vivida Pre-Whitening Toothpaste

- Green

Kulzer Vivida Home Whitening Syringe

- Transparent

Kulzer Vivida Tooth Conditioner

- White

Product Application

① Pre-treatment

Apply Kulzer Vivida Pre-Whitening Toothpaste to your teeth using your toothbrush. Brush your teeth thoroughly and make sure you reach every tooth. Take at least 2 to 3 minutes to brush your teeth. Then spit out the excess material, but do not rinse with water. Kulzer Vivida Pre-Whitening Toothpaste is not meant for regular use outside of the whitening treatment.

② Main treatment

Carefully break off the cap at the top of the Kulzer Vivida Home Whitening Syringe tooth whitening gel. Pour one drop of gel into the lower part of the whitening tray for each tooth. Please close the syringe with the inverted side of the cap after each treatment. Carefully place the prepared bleaching tray on your dental arch. Make sure that the tray is firmly in place and that the tooth whitening gel is evenly distributed. Please also take care not to overfill the tray so that no material gets onto the gums. Keep the tray in your mouth for a maximum of 15 to 30 minutes, depending on your dentist's instructions. Remove the bleaching tray and rinse your mouth thoroughly with water. No HPS tooth whitening gel should remain in your mouth. Then rinse the tray with cold water to clean it.

③ Product application – Follow-up treatment

Pour Kulzer Vivida Tooth Conditioner into the cleaned whitening tray. Place the tray on your dental arch again. Alternatively, you can carefully apply Kulzer Vivida Tooth Conditioner to all surfaces of your teeth with a toothbrush. Keep the Tooth Conditioner in your mouth for 5 to 10 minutes. If necessary, remove the tray and spit out the excess material. Do not rinse with water to maximise the effect of the Tooth Conditioner. Use the enclosed pictograms as a guide.

Tips on product use

For the dentist

- Inform the patient fully about the treatment procedure and its risks. The patient should not have any unanswered questions in this regard.
- Use on patients under the age of 18 is not permitted.
- For each application cycle, the first application must be carried out by a dentist or under their direct supervision. Thereafter, the product may be dispensed to the patient to continue treatment.
- Assess the patient's oral health and clarify whether the patient's expectations are realistic. Inform the patient that in a small percentage of patients the teeth will not lighten significantly.
- Check the patient's dental status and oral health to assess whether pre-treatment is indicated. All restorations should be intact or sealed with suitable temporary materials.
- If only minor discoloration is present on the tooth surface, such as after a professional cleaning, the frequency of Kulzer Vivida Pre-Whitening Toothpaste application can be adjusted according to individual needs.
- Inform the patient that temporary restorations may change colour during the bleaching process, while composite or polycarbonate materials, metals and ceramic restorations will not be affected. Existing restorations may need to be replaced after bleaching, as only natural teeth will become whiter.
- Determine the current tooth colour in order to document the success of the treatment. Photo documentation is recommended.
- Encouraging patients to incorporate the Kulzer Vivida Home Kit into their evening routine may enhance ease of use and overall convenience.
- When fabricating a bleaching tray, follow the general instructions of the manufacturer of the bleaching tray materials. Advise the patient not to smoke, eat or drink while wearing the bleaching tray.
- Advise the patient to avoid overfilling the tray, to wipe off excess whitening gel at the gum line and to rinse the mouth thoroughly with water after treatment with the whitening gel.
- Advise the patient to replace the cap on the syringe after each treatment to keep the whitening gel fresh.
- For patients requiring aesthetic or adhesive restorations, wait at least 2 weeks after the last whitening session to allow the colour to stabilise.
- Advise the patient of possible tooth sensitivity and irritated gums during treatment. The gums that come into contact with whitening gel may turn red or white, but will return to their normal colour after a few days.
- Inform the patient to discontinue treatment and contact you immediately if a problem occurs.
- The final colour of the teeth will be visible approximately 2 weeks after the start of treatment. Arrange a follow-up appointment after the treatment to check the result.

- After whitening the teeth, the patient should use Kulzer Vivida Tooth Conditioner. This helps to prevent and reduce possible tooth sensitivity.
- The patient should not eat, drink or smoke for at least 30 minutes after each treatment.
- Avoiding colouring foods and drinks such as coffee, tea, red wine and berries for 12 hours after each application is helpful to obtain the best possible result.

GENERAL INFORMATION

Please carefully observe the following safety instructions and all processing instructions in other sections. This product should only be used in accordance with the instructions for use. Any use not in accordance with these instructions is at the discretion and sole responsibility of the dental professional. Visually inspect for damage before use. Damaged products must not be used.

Warnings and safety instructions

Eyes and skin may become irritated. Avoid contact with skin. Sensitization through skin contact possible. If the product comes into contact with the skin, wash thoroughly with water immediately. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Consult an ophthalmologist if symptoms persist. Keep out of the reach of children.

Storage condition

- Do not use after the expiry date.
- After opening, use within 12 months.
- Store at temperatures between 4 °C and 25 °C (39 °F and 77 °F).
- Protect from sunlight.
- Store in a dry place.

Disposal information

Dispose of in accordance with local regulations.

Additional information

When giving feedback on the product, please always state the batch designation LOT and article number REF. Article number and batch designation: See information on syringe/tube/packaging.

Hydrogen peroxide $\leq 6\%$; Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.

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Kulzer GmbH