DIRECTIONS FOR USE	GB
IrriFlex [®]	
Endodontic needle for root-canal irrigation.	

Indication
Root-canal irrigation.

Properties
Flexible cannula with radial opening for liquid ejection along the canal walls.

Instructions for use

- Open the package by pulling along pre-cut notch provided for this purpose.
- Insert the «Secure Lock» part of the needle into the male nose of a previously filled Luer lock syringe and lock the needle by turning it.
- Make sure the needle is locked on the syringe.
- Use the syringe/needle assembly according to the usual protocol for canal irrigation.
- After use, disconnect the needle and dispose of it in accordance with local recycling regulations.
- Compatible fluids: Stabilised sodium hypochlorite 6%, EDTA 17%, Chlorhexidine digluconate 2%

Precautions
Check the length of the canal to be irrigated and use the 18-, 19-, 20- and 22 mm length markings to ensure proper positioning of the needle. Make sure that the needle can move freely to ensure constant flowback of the irrigation liquid.

Adverse reactions
None known.

Contraindications
None known.

Warning messages
Single-use device. Do not reuse.

Storage
Keep away from direct light.

Disinfection, cleaning and sterilisation
This product has been manufactured under clean-room conditions. It is ready for use. Do not sterilise before use.

Packaging	REF
Box of 5 needles	21403
Box of 20 needles	21400
Box of 40 needles	21401
Box of 100 needles	21402
Individual plastic packaging.	
Cardboard box of 5, 20, 40 or 100 individually wrapped needles.	

For dental professional use. Made in Switzerland.	
---	--

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© PD 200305